

NKANYWHERE™

SOFTWARE APPLICATION

Streamlined and Centralized
Device Management



Efficiency at Your Fingertips

- Easily apply updated alarm parameter default settings keeping with The Joint Commission's National Patient Safety Goal® to improve the safety of clinical alarm systems
- Implement timely updates to alarm settings across the Nihon Kohden Patient Monitoring System based on data from AWARE® Alarm Management and Reporting
- Rapidly deploy software upgrades and device settings on multiple devices concurrently, saving time and money
- Monitor the installation status for all supported devices on your network

System Requirements

The NKAnywhere™ software application deploys upgrades to compatible devices across the Nihon Kohden Patient Monitoring System. The software will need to be installed on a customer-provided laptop with sufficient processing power and memory.

System requirements for the NKAnywhere administrator:

- Windows® 7 or 10
- 2GB of memory or higher
- 2GB of available hard disk space or more

As a safety precaution, the NKAnywhere software application does not automatically update devices. Instead, an update must be accepted at the individual device level to prevent changes from taking place while a patient is actively being monitored.

NKANYWHERE™ SOFTWARE APPLICATION

Network Connectivity

The NKAnywhere administrator (admin) communicates with the agent that runs on the targeted devices via a computer network. The system utilizes a proprietary device discovery mechanism to allow the NKAnywhere admin to search for devices of interest on the network. Similarly, the device discovery mechanism also allows the device to dynamically and continuously report its existence to the NKAnywhere admin application.



Compatibility

NKANYWHERE-SUPPORTED DEVICES FOR UPDATING FIRMWARE

BSM-1700 Series (Ver. 01-15 or later)

BSM-3000 Series (Ver. 07-02 or later)

BSM-6000 Series (Ver. 07-02 or later)

CSM-1901 (Ver. 01-27 or later)

ORG-9100 Series (No minimum firmware version)

GZ-120P, GZ-130P, GZ-140P (Ver. 02-01 or later)

NKANYWHERE-SUPPORTED DEVICES FOR UPDATING SETTING DATA

BSM-1700 Series (Ver. 02-10 or later)

BSM-3000 Series (Ver. 07-41 or later)

BSM-6000 Series (Ver. 07-41 or later)

CSM-1901 (Ver. 01-27 or later)

ORG-9100 Series (No minimum firmware version)

GZ-120P, GZ-130P, GZ-140P (Ver. 02-01 or later)



**For more information, please contact us at
1-800-325-0283 or visit us.nihonkohden.com**

NIHON KOHDEN

**Exceptional Solutions,
Simply Delivered.™**

NKAnywhere is a trademark of Nihon Kohden. NK-HiQ is a trademark of Nihon Kohden Corporation. National Patient Safety Goals is a registered trademark of Joint Commission on Accreditation of Healthcare Organizations. AWARE is a registered trademark of Nihon Kohden. Windows is a registered trademark of Microsoft Corporation. Exceptional Solutions, Simply Delivered is a trademark of Nihon Kohden.

MMLB 248 (B)-CO-2743